# Advancis Medical

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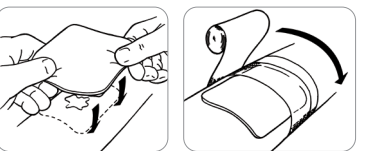
Medical

Medical

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland
	Device should not be cut		

## Advazorb® Silfix/Advazorb® Silfix Lite EN

Hydrophilic foam dressing with soft silicone wound contact layer

### Product description

A self adherent absorbent dressing with a Silfix® soft silicone wound contact layer. The dressing has a hydrophilic foam absorbent pad with high fluid handling capability. The soft silicone will adhere to surrounding dry skin but not to a wet wound and provides gentle release. This is an atraumatic dressing which means it is designed to minimise the pain and trauma associated with dressing change. The backing is breathable, prevents strike through and protects the wound.

### Indications

A wide range of exuding wounds including: Leg, foot and pressure ulcers, traumatic wounds, surgical wounds, abrasions and blisters.

### Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not use if allergic to silicone.

### Directions for use

Remove the clear plastic liners and place directly onto the wound ensuring the absorbent pad covers the entire wound area with an overlap of 1-2cm. Secure in place with an appropriate bandage or tape. Can be used under compression.

### Frequency of change

The dressing can be left in place for up to 7 days but should be changed when exudate is visible in the centre pad or the pad swells due to absorbed exudate.

### Warnings and Precautions

Advazorb® Silfix is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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