Brightwake Ltd, Lowmoor Business Park,  
Kirkby in Ashfield, Nottinghamshire, NG17 7JZ  
**Tel: +44 (0) 1623 751500**  
**Fax: +44 (0) 871 264 8238**  
**Email: info@advancis.co.uk**  
**Web: www.advancis.co.uk**

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# Alginate dressing impregnated with 100% Manuka honey

## Instructions for use

**Algivon®**

Alginate dressing impregnated with 100% Manuka honey

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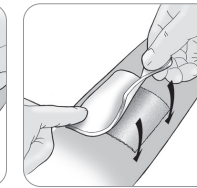
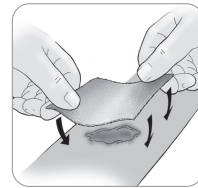
**pt BR**

#### Product description

Algivon is a non-adherent alginate dressing impregnated with Activon Manuka honey. It is effective against a broad spectrum of bacteria and effectively eliminates odours associated with malodorous wounds. Algivon effectively de-sloughs and de-bridges wounds and absorbs exudates whilst maintaining a moist wound healing environment and prolonging the retention of Activon Manuka honey at a wound site.

#### Indications

May be used on various wound types including superficial wounds, partial of full thickness wounds, sloughy wounds, necrotic wounds e.g pressure ulcers, leg ulcers, surgical wounds, burns, graft sites and infected wounds.

#### Directions for use

Ensure that the wound has been cleaned, gently dry the surrounding skin. Algivon® can be placed either side down directly onto the wound bed, it can be cut to size using sharp scissors, cover with a suitable secondary dressing. For larger wounds, several dressings can be placed side by side. Depending on the tissue type within the wound bed and level of exudate your secondary dressing of choice could be a foam dressing and/or bandage. In wounds with a high level of exudate an additional highly absorbent dressing can be introduced to help manage exudate.

#### Frequency of change

Depending on clinical observation, dressing may be left in place for up to 7 days. Algivon will be less effective as the honey is diluted by wound exudate, to maintain an effective level, change the dressing when the colour changes significantly.

#### Contra-indications

Contains glucose. Monitor blood glucose levels in patients with diabetes. Do not use if allergic to bee venom/ honey. A few patients may experience pain when applied; depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. If pain continues unabated, discontinue use and irrigate the wound with saline solution.

#### Storage

Store at room temperature.

#### Disposal

Dispose of hygienically after use. This is a single-use medical device. Re-use of this Algivon dressing after being placed on the patient may result in patient infection and/or cross-contamination. Sterilisation, reprocessing, cleaning and/or disinfection, may also compromise the product characteristics, resulting in trauma or infection to the patient.

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	REF	Catalogue Number		Consult instructions for use
		Do not re-use		Keep dry
	STERILE	Sterilized using irradiation		Keep away from sunlight
		Do not use if package is damaged		Manufacturer
		Caution		Authorized representative in the European Community
		Quantity		Batch code
	UDI	Unique Device Identifier (UDI carrier including the AIDC and human readable information)		Medical Device
		Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
		The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
		Do not re-sterilize		Single sterile barrier system
		Use-by date		Indicates the authorised representative in Switzerland