Advancis Medical

 Brightwake Ltd, Lowmoor Business Park, Kirkby in Ashfield, Nottinghamshire, NG17 7JZ
Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238

Email: info@advancis.co.uk

Web: www.advancis.co.uk

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Utylizacja

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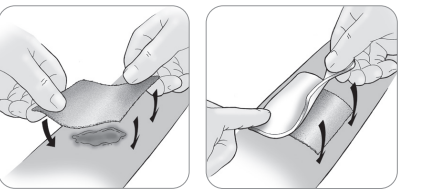
Algivon® Plus

Reinforced alginate dressing impregnated with 100% Manuka honey

Instructions for use

Advancis Medical

REF	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
STERILE	Sterilized using irradiation		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity	LOT	Batch code
UDI	Unique Device Identifier (UDI carrier including the AIDC and human readable information)	MD	Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland

Algivon® Plus

Reinforced alginate dressing impregnated with 100% Manuka honey

Product description

Algivon® Plus has reinforced alginate fibres enabling a sustained, slow release of honey whilst maintaining the integrity of the dressing. Algivon® Plus is an ideal choice for wetter wounds as the alginate has a small capacity to absorb, meaning the honey isn't washed away with exudate. The dressing is very soft and conformable, ideal for debriding and de-sloughing large areas of necrotic and sloughy tissue.

Indications

Algivon® Plus may be applied to any wound but especially: Pressure ulcers, leg ulcers, diabetic ulcers, surgical wounds, burns, graft sites and infected wounds.

Contra-indications

Although the honey is not absorbed into the blood stream, we advise monitoring the blood glucose levels in patients with diabetes. Do not use if allergic to bee venom. Discomfort can be experienced when honey is applied, depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. The initial discomfort usually subsides, however if it does continue, discontinue use and irrigate the wound with saline solution.

Directions for use

Remove the clear liners on each side of the dressing and place directly over the wound. Algivon® Plus is placed either side down onto the wound surface; dressings can be placed side by side to cover large wound areas or cut to size ensuring that sharp scissors are used. Apply a suitable secondary dressing to manage exudate.

Warnings and Precautions

Algivon® Plus is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

Frequency of change

Depending on wound exudate levels, any surrounding interstitial fluid, oedema and dressing regime may require changing daily but may be left in place for up to 7 days. Will be less effective as the honey is diluted by wound exudate, to maintain an effective level change the dressing when the colour changes significantly.

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