ADVANCIS

 Brightwake Ltd, Lowmoor Business Park,
Kirkby in Ashfield, Nottinghamshire, NG17 7JZ
Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk

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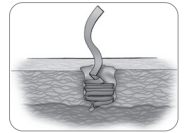
ADVANCIS

ALGIVON PLUS RIBBON

Reinforced alginate ribbon Impregnated with 100% Manuka Honey

Instructions for use

Algivon® Plus Ribbon

Reinforced alginate ribbon impregnated with 100% Manuka honey

Product description

Algivon® Plus Ribbon has reinforced alginate fibres enabling a sustained, slow release of honey whilst maintaining the integrity of the dressing. It is an ideal choice for wetter wounds as the alginate has a small capacity to absorb. The ribbon is very soft and conformable, shaped for easy application within cavities and sinuses meaning the wound can be easily packed for direct contact with the entire wound site.

Indications

Algivon® Plus Ribbon may be applied to any wound but especially: Cavities, sinuses, pressure ulcers, leg ulcers, diabetic ulcers, surgical wounds, burns, graft sites and infected wounds.

Contra-indications

Although the honey is not absorbed into the blood stream, we advise monitoring the levels of blood glucose in patients with diabetes. Do not use if allergic to bee venom. Discomfort can be experienced when honey is applied, depending on sensitivity of the wound it may be necessary to consider an appropriate level of analgesia. The initial discomfort usually subsides, however if it does continue, discontinue use and irrigate the wound with saline solution.

Directions for use

Pack Algivon® Plus Ribbon into a deep wound, cavity or sinus; ribbon can be cut to size ensuring that sharp scissors are used. Do not pack cavity too tight. Apply a suitable secondary dressing to manage exudate. Wound Probe: do not insert into cavities smaller than the probe.

Frequency of change

Depending on wound exudate levels, any surrounding interstitial fluid, oedema and dressing regime may require changing daily but may be left in place for up to 7 days. Will be less effective as the honey is diluted by wound exudate, to maintain an effective level change the dressing when the colour changes significantly.

Warnings and Precautions

Algivon® Plus Ribbon is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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