 Brightwake Ltd, Lowmoor Business Park, Kirkby in Ashfield, Nottinghamshire, NG17 7JZ
Tel: +44 (0) 1623 751500
Fax: +44 (0) 871 264 8238
Email: info@advancis.co.uk
Web: www.advancis.co.uk

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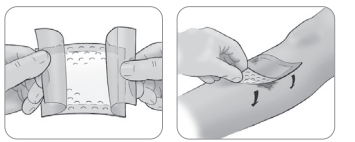
PL Eclipse® Adherent

Eclipse® Adherent

Super absorbent dressing with soft silicone contact layer

Instructions for use

Eclypse® Adherent

Superabsorbent dressing with Silfix® soft silicone contact layer

Product Description

Eclypse® Adherent is an exudate management wound dressing. The dressing combines a Silfix® soft silicone contact layer, a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehisced wounds, fungating wounds and donor sites.

Contra-Indications

Arterial bleeds and heavily bleeding wounds.

Directions for Use

Apply to wound (white silicone face down, beige facing up) allowing a minimum of a 2cm border overlap around the wound area. Do not cut Eclypse® Adherent. Dressings can be placed side-by-side to cover large wounds. Secure the dressing pad with tape or bandage appropriately. Eclypse® Adherent may be used under compression bandages. Eclypse® Adherent can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclypse® Adherent must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Disposal

Dispose as clinical waste.

Use-by date

Indicates the authorised representative in Switzerland

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