ADVANCIS

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Super Absorbent Multisite Dressing

Instructions for use

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Moderate/High Exudate
	Device should not be cut		

Eclypse® Contour

Superabsorbent multisite dressing

Product description

Eclypse® Contour is an exudate management wound dressing especially designed to fit to body contours. The dressing combines a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehisced wounds, fungating wounds and donor sites.

Contra-indications

Arterial bleeds and heavily bleeding wounds.

Directions for use

Eclypse® Contour is placed white face down on the wound surface, with beige backing uppermost. For larger wounds place several dressings side by side. Protect friable skin, or drier areas of the wound with a non-adherent dressing under Eclypse® Contour. Do not cut Eclypse® Contour. Secure the dressing pad with tape or bandage appropriately. Eclypse® Contour may be used under compression bandages. Eclypse® Contour can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclypse® Contour must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

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