Eclipse® Foot

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Superabsorbent foot wrap dressing

Product description

Eclipse® Foot is an exudate management wound dressing especially designed for the foot. The dressing combines a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate, a sub-backing layer for even exudate distribution and a non-strikethrough backing to prevent leakage.

Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers and diabetic foot ulcers.

Contra-indications

Arterial bleeds and heavy bleeding wounds.

Directions for use

Open out the dressing fully, white side up placing the end of foot in the middle of the dressing so the heel is at the edge. Fold the side panels over the foot one at a time and secure with tape. Do not cut Eclipse® Foot. Eclipse® Foot can be used under compression bandaging. Eclipse® Foot can be used as a primary or secondary dressing.

Frequency of change

Wear time will depend on the level of exudate. Eclipse® Foot must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

Disposal

Dispose as clinical waste.

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI) carrier including the AIDC and human readable information		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland
	Device should not be cut		Moderate/High Exudate

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