


New!

# Eclipse NON-BACKED


## Non-backed superabsorbent dressing

### Folding Guide


#### In between fingers and toes:

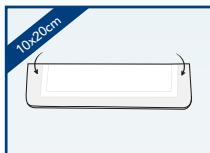


Fold the dressing in half lengthways

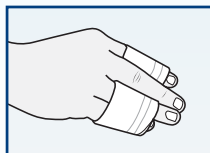


Weave between fingers or toes

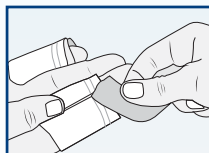


Secure with tape

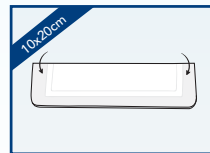

Fold the dressing in half lengthways

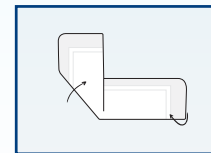


Fold the corners over to create a v-shape

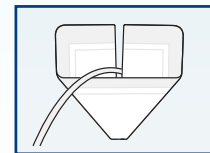


Turn over and position around the PEG site


#### For heel or elbow:

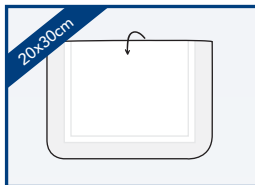


Fold the dressing in half lengthways

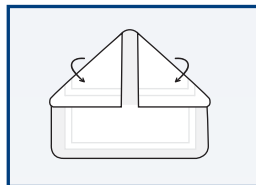

Fold each corner of the folded edge inwards, leaving a gap in the middle

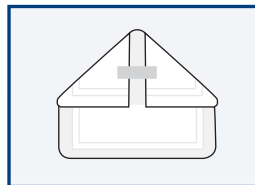

Secure the folded corners with tape

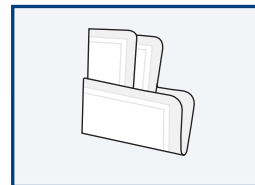

Open out the dressing into a cup shape



Put on heel/elbow/shoulder

# Eclipse NON-BACKED

## Non-backed superabsorbent dressing

Eclipse® Non-Backed is the latest addition to the Eclipse range. It is a double-sided superabsorbent dressing with highly absorbent CrystaLock™ Technology to lock away exudate, bacteria and MMPs.<sup>1</sup> It's unique central rapid wicking layer\* helps even distribution of exudate around the dressing.

### Ordering Information

Stock Code	Size	Pack Size
CR4516	10 x 10cm	20
CR4517	10 x 20cm	10
CR4518	20 x 20cm	10
CR4519	20 x 30cm	20

\*Except 10x10cm

#### Reference

1. Weigand, C. White, R. (2013) Binding and inhibition of protease enzymes, including MMPs, by a superabsorbent in vitro. Journal of Wound Care Vol 22, No 5

### Get in touch

 +44 (0)1623 751 500

 [info@advancis.co.uk](mailto:info@advancis.co.uk)

### Find us online



[www.advancis.co.uk](http://www.advancis.co.uk)