## Eclipse® Non-Backed

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	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/ Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		Indicates the authorised representative in Switzerland
	Device should not be cut		

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Non-Backed Superabsorbent dressing

### Product description

Eclipse® Non-backed is an exudate management wound dressing. The dressing combines a rapid wicking layer, a CrystaLock™ layer, which is highly absorbent and locks away exudate and a sub-backing layer for even exudate distribution.

### Indications

Moderate to heavily exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers, sloughy wounds, granulating wounds, lymphatic legs, post-operative and dehiscent wounds, fungating wounds and donor sites.

### Contra-indications

Arterial bleeds and heavily bleeding wounds.

### Directions for use

Eclipse® Non-backed is placed face down on the wound surface, with either face uppermost. For larger wounds place several dressings side by side. Protect friable skin, or drier areas of the wound with a non-adherent dressing under Eclipse® Non-backed, Do not cut Eclipse® Non-Backed dressings. Secure the dressing pad with tape or bandage appropriately. Eclipse® Non-backed may be used under compression bandages. Eclipse® Non-backed can be used as a primary or secondary dressing.

### Frequency of change

Wear time will depend on the level of exudate. Eclipse® Non-backed must be changed before dressing reaches capacity but depending on wound exudate levels, may be left in place for up to 7 days.

### Warnings and Precautions

Eclipse® Non-backed is a single use product. If used on more than one patient, cross-contamination or infection may result. Opening the dressing pack compromises the sterile barrier. Any unused dressings should be discarded.

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Brightwake Ltd, Lowmoor Business Park,
CS Lifesciences Limited, The Black Church,
Swiss AR Services GmbH