
# Hemosep

Revolutionary Cell Salvage

## Operational Manual

Hemosep® Shaker Unit - CR4213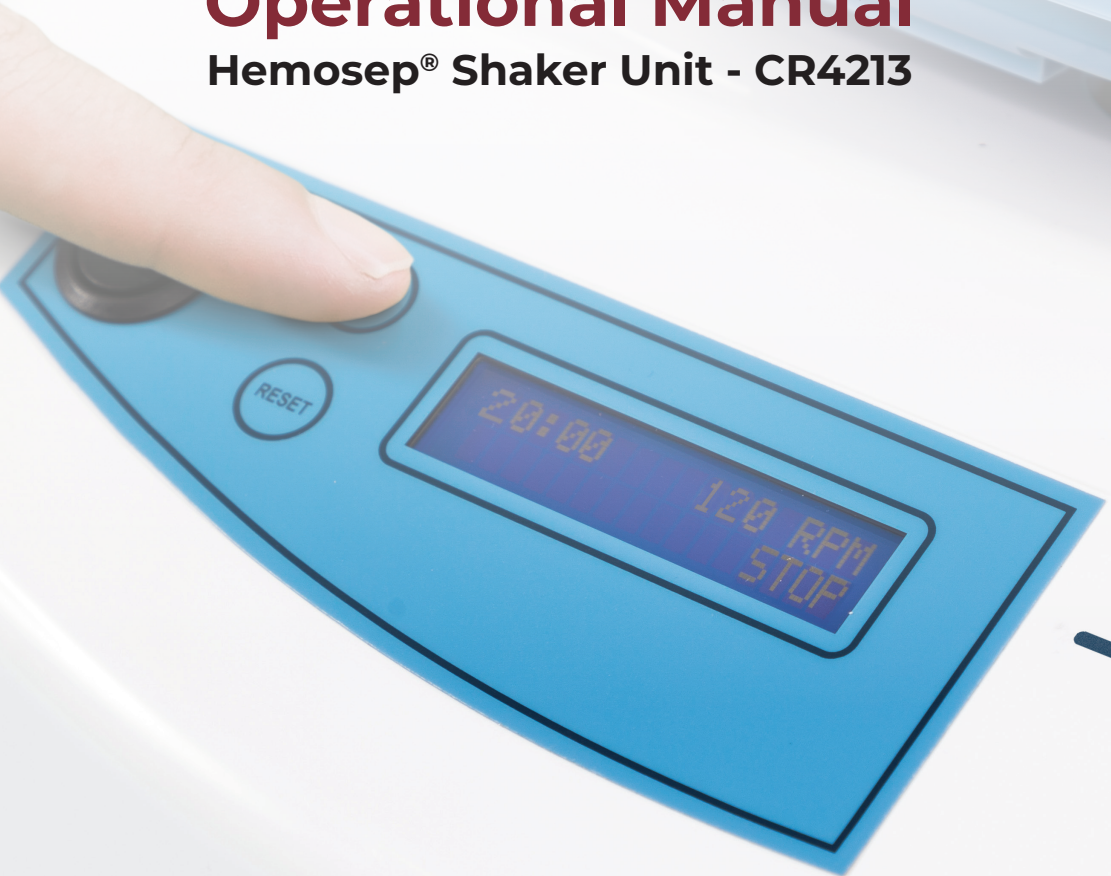

## Contents Of Carton:

1. Hemosep<sup>®</sup> Shaker Unit
2. External bench top power supply, DC 12V, 3.2A- XP Power, AFM60US12
3. Operating manual
4. Power supply unit (with 3 x power leads for different regions/ territories) DW01

**NOTE:** Remove all packing materials carefully, and retain them for future use.

## Safety:

- Use only as specified by the operating instructions in this Operational Manual or the intrinsic safety of the device could be impaired.
- Connect only to a power supply that was delivered with the device, if there is a problem with the power check with the manufacturer.
- Do not operate the device outside the recommended environment (theatre).
- Do not impede the movement of the Hemosep<sup>®</sup> Shaker Unit carriage; always keep clear space around the device.
- This product is designed for use in a static controlled environment.
- The device should be protected from shocks or falls.
- If fluid is spilt inside the case, disconnect it from the power supply and have it checked by a competent person.
- The Hemosep<sup>®</sup> Shaker Unit and power supply should be mounted on a horizontal, stable work surface with an area of 200mm of free space.
- Do not exceed the load rating (5Kg) on the shaker including the carriage.
- The device should only be transported in a horizontal position and packed in the packaging it was delivered in or other similar quality packaging.
- Portable and mobile RF communications equipment can affect Medical Electrical Equipment.
- The device must only be operated by trained personnel.
- Do not position the Hemosep Shaker Unit so that it is difficult to operate the external power supply.

**WARNING:** Finger trap / Crush hazard beneath carriage tray. Avoid contact during operation.

Ensure this device functions as intended when in close proximity to other electrical equipment.

## Device Accessory List

In order to operate the Hemosep<sup>®</sup> correctly, the Hemosep<sup>®</sup> system comprises of the following parts;

Hemosep<sup>®</sup> Cell Concentrator Pack, Hemosep<sup>®</sup> Shaker Unit, Hemosep<sup>®</sup> Intra-Operative stand, Hemosep<sup>®</sup> Intra-Operative Kit.

### Order code(s):

Hemosep<sup>®</sup> Cell Concentrator Pack – CR4426

Hemosep<sup>®</sup> Shaker Unit – CR4213

Hemosep<sup>®</sup> Intra-Operative stand – CR4429

Hemosep<sup>®</sup> Intra-Operative Kit- CR4428

Hemosep<sup>®</sup> One to Three Connector – CR4432

Hemosep Suction Kit - CR4434

 Do not use if package is damaged

## Specifications:

The Hemosep<sup>®</sup> Shaker Unit is designed to agitate a blood sample which is contained within the Hemosep<sup>®</sup> Cell Concentrator Bag. This unit is designed for use only within a controlled theatre environment.

The function settings of the machine are set and cannot be changed.

<b>Rotational operation:</b>	Orbital reversing
<b>Rotational speed:</b>	120 rpm
<b>Run time operation:</b>	Can be powered continuously, program runs for 20 minutes.
<b>Orbit:</b>	20mm diameter
<b>Maximum load:</b>	5Kg
<b>Power Supply:</b>	External bench top power supply, DC 12V, 3.2A- XP Power, AFM60US12 (With Tray) 340 X 380 X 235mm
<b>Device dimensions:</b>	
<b>Weight (with tray):</b>	Not more 20Kg
<b>Noise level:</b>	Less than 70 dB
<b>Environmental Operating Conditions:</b>	15 - 40°C 10 - 80% RH
<b>Protection class:</b>	Class 1 ME Equipment
<b>Note:</b>	Power supply's specified as part of Class 1 ME equipment

## Guarantee:

When used in accordance with the conditions stated in the general information and within the parameters detailed in the specifications, this product is guaranteed for one year against faulty materials or workmanship.

## Service:

There are no user serviceable parts inside the unit. All faulty products to be returned to Advancis Surgical or our distributor.

 **WARNING:** No modification of this equipment is allowed

## Maintenance:

Maintenance should be conducted in line with manufacturer's specification. Do not attempt to repair. If a fault should occur the unit should be packaged in the original packaging and returned to your local distributor. Power supply should be checked routinely for degradation, and if found the unit must not be used.

## Disposal:

For the Disposal of the device consult Advancis Surgical.

## Storage And Transport

**Humidity limitation**

**Temperature range**

## Instructions For Use

1. Place the Hemosep® Shaker Unit on a flat surface with 200mm clearance around the unit (see figure 1).
  2. On first use, attach the Intra-Operative stand base plate to the adapter on the shaker unit using the Allen bolts and Allen key provided. On subsequent uses, check the base plate is still securely attached.
  3. Attach the Hemosep® blood collection reservoir clamp to the IV pole on the Intra-Operative stand. Make sure this is secure. It should be positioned so the bottom of the clamp is slightly higher than the lid of the Hemosep® Shaker Unit.
  4. The Intra-Operative stand can be left in place for subsequent use.
  5. Connect the external power supply to the appropriate power lead for your territory.
  6. Plug the power lead into the plug socket and the small round plug into the rear of the Hemosep® Shaker Unit (see figure 2).
  7. Turn the plug socket on.
- ⚠ Warning:** Do not touch the patient while connecting the equipment to the power supply unit.
8. At the front of the Hemosep® Shaker Unit there is a BLACK power ON/OFF button, switch this to the ON position (see figure 3).
  9. The display screen will indicate the RUN time (upper left of screen) and speed RPM (upper right of screen). The display will also indicate START below the RPM display (bottom right of screen) (see figure 4).
  10. Open the top section of the tray (see figure 5).
  11. Place the Hemosep® Cell Concentrator Bag on the top section, making sure the tubes are free and the bag is located on the location pins (see figure 6).
  12. Close the top of the carriage tray so that it locks into the lower tray. Shut the locking clips to ensure it is secure.
  13. Once the Hemosep® Cell Concentrator Bag is set in place press the RUN/STOP button on the front to start the Hemosep® Shaker Unit running.
  14. Once the Hemosep shaker unit has run for 20 minutes, it will automatically stop and the timer will reset.
  15. The Hemosep® Shaker Unit can be stopped earlier by manually pressing the RUN/STOP button.


16. To reset the clock, press the RESET button once the Hemosep® Shaker Unit has stopped.

**CAUTION:** If at any time there is power loss to the Hemosep® Shaker Unit or the Reset button is pressed whilst it's in use the run time will automatically reset back to 20 minutes

**CAUTION:** This product is designed for use in a static controlled environment. If subjected to static discharge, the unit may stop working, in which case it can be re-set by switching the mains supply off and then on again. Safe termination of the operation of the Hemosep® Shaker Unit is by disconnection of the power supply.

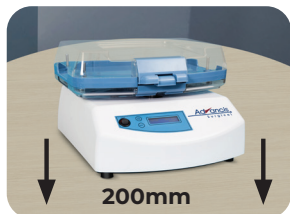
**Fig. 1**

**Fig. 2**

**Fig. 3**

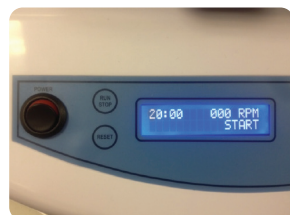**Fig. 4**

**Fig. 5**

**Fig. 6**

## Cleaning:

Only cleaning liquids that do not contain concentrated organic solvents, alkali or acid can be used for cleaning the Hemosep® Shaker Unit.

## Cleaning Process:

1. Turn off the Hemosep® Shaker Unit at main supply.
2. Make sure you have correct cleaning liquids and cleaning wipes required.
3. Spray Hemosep® Shaker Unit or wipes with cleaning liquid.
4. Gently wipe Hemosep® Shaker Unit all over with the wipes (repeat step 3 to 4 until clean).
5. Wipe Hemosep® Shaker Unit down with dry wipes.
6. Spray Hemosep® Shaker Unit carriage tray or wipe with cleaning liquid.
7. Wipe Hemosep® Shaker Unit carriage tray all over with wipes (repeat step 6 to 7 until clean).
8. Wipe Hemosep® Shaker Unit carriage tray down with dry wipes.

## System Display

Ready for start. Press RUN to begin.

During run, time counts down, and the RPM is displayed. Press STOP at any time to stop the shaker.

SLOWING DOWN displayed when STOP is pressed during a run.

When a run is interrupted, press RESET to start cycle again, or START to continue the cycle.

	<p><b>Warning/ Caution</b></p>
	<p><b>Product is not made with natural rubber latex</b></p>
	<p><b>Consult instructions for use</b></p>
	<p><b>Catalogue reference</b></p>
	<p><b>Manufacturer</b></p>
	<p><b>CE Mark</b></p>
	<p><b>Date of manufacture</b></p>
	<p><b>Serial number</b></p>
	<p><b>Humidity limitations</b></p>
	<p><b>Temperature range</b></p>

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Advancis Surgical is a trading name of Brightwake Ltd.