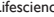
# Advancis Medical

 Brightwake Ltd, Lowmoor Business Park, Kirkby in Ashfield, Nottinghamshire, NG17 7JZ  
**Tel:** +44 (0) 1623 751500  
**Fax:** +44 (0) 871 264 8238  
**Email:** info@advancis.co.uk  
**Web:** www.advancis.co.uk

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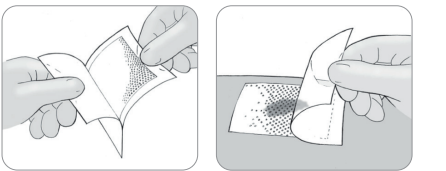
## Instructions for use

# Advancis Medical

	Catalogue Number		Consult instructions for use
	Do not re-use		Keep dry
	Sterilized using ethylene oxide		Keep away from sunlight
	Do not use if package is damaged		Manufacturer
	Caution		Authorized representative in the European Community
	Quantity		Batch code
	Unique Device Identifier (UDI carrier including the AIDC and human readable information)		Medical Device
	Date of manufacture/Country of manufacture		CE Mark for Northern Ireland, EC & EU
	The product has made a financial contribution towards the recovery and recycling of packaging in Europe		Importer
	Do not re-sterilize		Single sterile barrier system
	Use-by date		

## Silflex

Soft silicone wound contact layer

### Product description

A wound contact layer made from polyester mesh and coated with Silfix soft silicone. Silflex is designed to gently adhere to the skin surrounding a wound and not to the wound bed. This is an atraumatic dressing designed to minimise the pain and trauma associated with dressing change. The dressing's large perforations allow for the passage of exudate and irrigation of wound.

### Indications

Skin tears, skin abrasions, surgical wounds, second degree burns, lacerations, leg and pressure ulcers.

May be used under Negative Pressure Wound Therapy.

### Directions for use

Remove the clear liners on each side of the dressing and place directly over the wound. Silflex may overlap the wound edges. Cover with a secondary dressing of choice, we advise to use the Advazorb Lite range for low exuding wounds and Advazorb or Eclipse range for medium to high exuding wounds. The Silflex dressing can be cut to size ensuring sharp, sterile scissors are used.

In the presence or suspicion of infection, Activon Tube may be used to assist in treatment.

### Contra-indications

Do not use if allergic to silicone. The wound bed can become too moist as a result of poor secondary dressing choice, which may lead to hypergranulation. Manufacturer's recommendation of secondary dressings should be followed. When using under NPWT, the dressing should be changed between 24-72 hours according to the treating healthcare professional's assessment.

### Frequency of change

The dressing may be left in place for up to 14 days depending on the wound condition which will be assessed by the Healthcare Professional.

### Disposal

Dispose as clinical waste in accordance with local regulations.

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